

AED post-use instructions

These instructions are to be followed after the AED has been disconnected from the patient and the patient care is no longer being rendered on scene. The AED should be returned to service as soon possible.

1. Discard any of the following that have used; electrode pads, gloves, razor, and towlettes.
2. If necessary, clean machine and/or case by gently wiping with damp cloth.
3. Wash your hands thoroughly.
4. If you feel that you may have been exposed to an infectious disease via patient contact, notify the responding Clackamas Fire crew before they depart the scene so they can initiate exposure procedures.
5. Notify Clackamas Fire EMS Division (Division Chief Bill Conway at 503-742-2645 or email William.Conway@Clackamasfire.com) that an AED was used in an emergency situation. Clackamas Fire will provide a loaner AED and return your AED to working condition in exchange for incident data from the AED.